



CORPORATE OFFICE
2691 Saturn Street | Brea, CA 92821
Tel: (714) 524-0663 | Fax: (714) 364-8113



ST&G CORPORATION
Rm. 911 43, Yangsan-ro
Yeongdeungpo-gu, Seoul, Korea



MDSS GmbH

Schiffgraben 41 | 30175 Hannover, Germany



REF : 1255S

3 Prong Pyramid Connector SS

Weight: 127 grams

Weight Limit: 125 kg/ 275 lbs



1255S

1255S IFU REV. C (02-18-21)

1



CORPORATE OFFICE
2691 Saturn Street | Brea, CA 92821
Tel: (714) 524-0663 | Fax: (714) 364-8113



ST&G CORPORATION
Rm. 911 43, Yangsan-ro
Yeongdeungpo-gu, Seoul, Korea



MDSS GmbH

Schiffgraben 41 | 30175 Hannover, Germany



REF : 1255S

3 Prong Pyramid Connector SS

Weight: 127 grams

Weight Limit: 125 kg/ 275 lbs



1255S

1255S IFU REV. C (02-18-21)

1

1. Application

The 1255S, 3 Prong Pyramid, SS is to be exclusively used for the prosthetic fitting of lower extremity amputations.

2. Description

The 1255S, 3 Prong Pyramid, SS is integrated into the prosthetic socket's laminate as lamination anchor. This adapter can be used for fitting of transtibial or transfemoral amputees. The four set screws are used for anchoring the part to the mating pyramid system as well as for making angular adjustment for alignment. Apply loctite to all screws once the alignment and connection is finalized.

3. Reinforcement Recommendations

The type of reinforcement used depends on the type of loading expected. It is up to the clinician to determine adequate layup and reinforcement for the given patient usage and activity level.

4. Warranty

Warranty applies only when the product is used as intended without modification, following all manufacturer's recommendations. ST&G offers 2 years warranty.

5. Declaration of Conformity

ST&G as manufacturer declares that the 1255S, 3 Prong Pyramid, SS conforms to the requirements of 93/42/ EEC Guideline.

Notice

Tighten all the screws, which are side by side, using a torque wrench.

1. Application

The 1255S, 3 Prong Pyramid, SS is to be exclusively used for the prosthetic fitting of lower extremity amputations.

2. Description

The 1255S, 3 Prong Pyramid, SS is integrated into the prosthetic socket's laminate as lamination anchor. This adapter can be used for fitting of transtibial or transfemoral amputees. The four set screws are used for anchoring the part to the mating pyramid system as well as for making angular adjustment for alignment. Apply loctite to all screws once the alignment and connection is finalized.

3. Reinforcement Recommendations

The type of reinforcement used depends on the type of loading expected. It is up to the clinician to determine adequate layup and reinforcement for the given patient usage and activity level.

4. Warranty

Warranty applies only when the product is used as intended without modification, following all manufacturer's recommendations. ST&G offers 2 years warranty.

5. Declaration of Conformity

ST&G as manufacturer declares that the 1255S, 3 Prong Pyramid, SS conforms to the requirements of 93/42/ EEC Guideline.

Notice

Tighten all the screws, which are side by side, using a torque wrench.